



User guide Stirex pliers

Art. no: P-91B, P-91BS, P-91BM, P-90B, P-90BS, P-90BM, P-1B, P-2B, P-1L, P-2L, P-20B, MT-70

Safety:

- Do not use the product for any other purpose other than cutting light material.
- Please note that the front is sharp.
- These pliers should be kept away from children at all times as they are not suitable for children to use. The pliers should be used under supervision of an adult.
- Avoid using these pliers for squeezing hard materials.
- If you should feel uncomfortable or feel pain while using the plier please stop immediately.

Usage:

Stirex pliers all have the same area of usage as normal pliers. All our pliers are light, reliable and comfortable with a self-opening handle which is working in a soft and smooth way and with low spring pressure. Observe that no distal finger control is required. The front, made from stainless steel, is cast in handles of environmental friendly plastic. All Stirex pliers are only suitable for light use.

Maintenance: Machine washable and boil resistant.
Hand wash best for gentle washing.

Technical specification:

Art no:	Name:	Length of front:	Weight:
P-91B	Stirex Pick up plier	6,5 cm	29 g
P-91BS	Stirex Pick up plier	6,5 cm	29 g
P-91BM	Stirex Pick up plier	6,5 cm	39 g
P-90B	Stirex Pick up plier	7 cm	27 g
P-90BS	Stirex Pick up plier	7 cm	27 g
P-90BM	Stirex Pick up plier	7 cm	37 g
P-1B	Stirex Plier	4,5 cm	26 g
P-2B	Stirex Plier	4,5 cm	27 g
P-1L	Stirex Plier	4,5 cm	28 g
P-2L	Stirex Plier	4,5 cm	28 g
P-20B	Stirex Plier	4,5 cm	20 g
MT-70	Stirex Pick up plier	12 cm	104 g

Material:

Handle: PP, **Front:** Steel

Symbols:



Our products meet the requirements for CE-marking and follow the EU-directives of 93/42/EEG and 2007/47/EG for medical devices.

Manufacturer:

Stirex Innovation AB
Box 109
191 22 Sollentuna, Sweden
Phone: +46-8-760 02 55
www.stirex.se
info@stirex.se